



Cobas h 232 Quiz CDHA

Capital Health

Final Approval: Dr Bassam Nassar

Name: _____

Site: _____ Unit: _____

Trainer: _____ Date: _____

1. When cleaning your Cobas h 232 meter you should:
 - a. Wipe all exterior surfaces with a lint-free cloth moistened in 10% bleach solution
 - b. Wipe easily accessible areas with a lint-free cloth moistened in 10% bleach solution
 - c. Spray meter with 10% bleach solution and dry with lint free cloth
 - d. A and B

2. Proper insertion of test strip is as follows:
 - a. With the application and test area are facing up
 - b. With the barcode facing up
 - c. Slide the test strip in as far as it will go
 - d. A and C

3. Which of the following factors influence when you should perform an IQC control test?
 - a. Once every 24 hours.
 - b. If meter is dropped.
 - c. When troubleshooting the system.
 - d. All of the above.

4. Which of the following samples are used for testing on the h 232 meter?
 - a. EDTA venous whole blood
 - b. Citrate venous whole blood.
 - c. Heparinized venous whole blood.
 - d. All of the above.

5. The Cardiac Risk associated with a Troponin result of < 50 ng/L is:
 - a. Low
 - b. Medium
 - c. High

6. If a patient result does not correlate with a patient's symptoms you should repeat testing for confirmation
 - a. True
 - b. False

7. After removing the test strip from the foil pouch you must insert into meter.
 - a. Within 15 minutes.
 - b. Within 30 minutes.
 - c. Within 60 minutes.
 - d. Within 90 minutes.

8. Prior to performing testing on the Cobas h232 you must enter your Operator ID?
 - a. True
 - b. False



Cobas h 232 Quiz CDHA

Capital Health

Final Approval: Dr Bassam Nassar

Answer Key:

1. d)
2. d)
3. d)
4. c)
5. a)
6. a)
7. a)
8. a)