

TITLE: QE BT BloodTrack Courier Emergency Blood Job Aid	Doc #: 30143
Section: Management System\PLM\Blood Transfusion Services\HemoSafe \	Version: 2.0 Current
Document Owner: BT Technical Specialist (MG)	Effective Date: 2018/02/26
Final Approval: Robert S Liwski	

Physician requests Unmatched Red Cells

Unmatched Blood to be transfused now.

1. Login
2. Select **Emergency Blood.**
3. Tap either O Neg or O Pos as ordered by the attending physician.
4. Enter patient's **FULL 10-digit MRN** (which may not bring up any patient info other than the MRN)
Message 'invalid number' means that you did not enter the full 10 digits
5. Confirm correct patient info.
6. **Wait** until door is unlocked then push and **HOLD** the door open and remove the unit.
7. **Scan** the unit number when prompted.
8. Answer question: "Visual Inspection OK?"
9. **Retrieve** labels from HemoSafe printer.
10. Answer question: "Did the Compatibility label print OK?"
11. Check and **stick** the label on **BOTH** copies of the tag. **DO NOT APPLY LABEL TO BAG!**

Unmatched in a Cooler or Group Specific Unmatched is desired.

1. Call BTS to have unmatched blood dispensed in cooler or to the backup fridge.

NOTE: If labels do not print, ensure Printer is not paused.
The LED light must be solid green. If flashing green then press the button and all your labels will print. If the LED light is red then the lid is not closed properly on the printer.

Referenced to:

Document Name	Document #	Document Location
QE BT BloodTrack Enquiry and HemoSafe Usage Procedure	24764	Paradigm BTS Website
QE BT BloodTrack Enquiry and HemoSafe Usage Training Record Form	25982	Paradigm